


Comfortable, **easy to apply** and **remove**

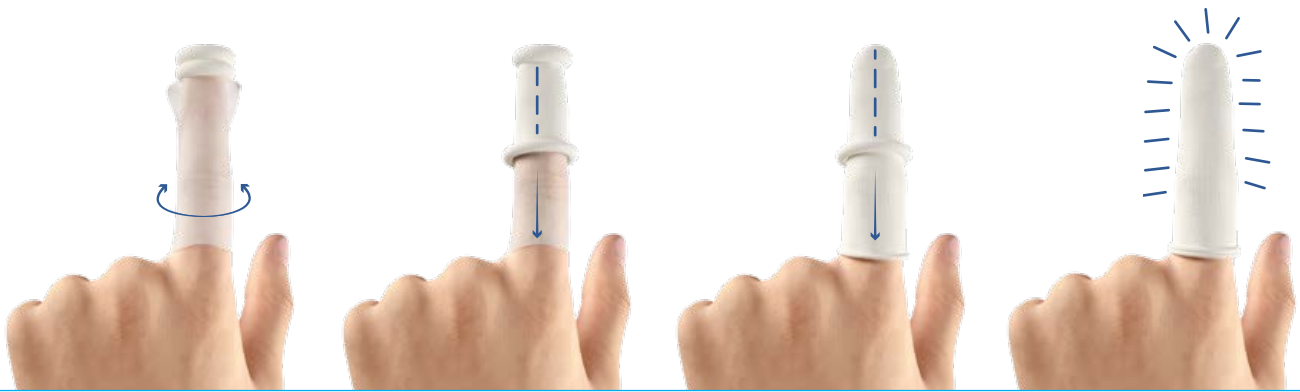

WHAT IS IT?

ADAPTIC™ DIGIT Non-Adhering Digit Dressing is made of a unique, tacky, silicone-coated fabric covered with a tubular bandage. It is specifically designed for dressing fingers and toes.



Comfortable

ADAPTIC™ DIGIT Dressing does not adhere to the wound. It is conformable and non-bulky, therefore allowing maximum digit flexibility.



Easy to apply and remove


ADAPTIC™ DIGIT Dressing is quick and simple to apply; no applicator or scissors are required. It removes cleanly from the wound, minimizing trauma¹.

USER GUIDE

Before application

Refer to the instructions for use. Select a size of ADAPTIC™ DIGIT Dressing according to the size of the finger or toe. Prepare the wound according to appropriate wound management protocol. Ensure skin surrounding the wound is dry. Remove the ADAPTIC™ DIGIT Dressing from the box.

Dressing preparation

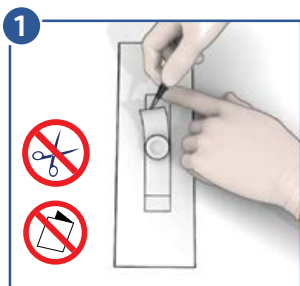


1. Using aseptic technique, peel apart the two sides of the packaging.
2. Remove **ADAPTIC™ DIGIT Dressing** from shiny backing strip.

Dressing application



1. Place primary silicone non-adhering layer over digit tip and gently smooth over any broken skin.
2. Roll down lower ring of tube bandage over primary strip (and secondary dressing if required).
3. Follow this with the top ring to form a double layered tube for protection.

Dressing change and removal

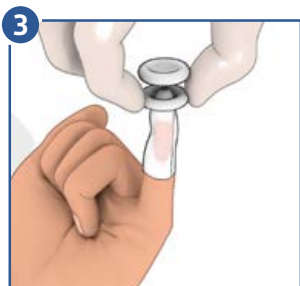

- Dressing change frequency is dictated by good wound care practice and will depend on the condition of the wound. **ADAPTIC™ DIGIT Dressing** may be left in place for several days.
1. Gently roll up the upper ring of the tubular bandage and repeat with the other ring.
 2. Gently remove the non-adhering layer from the wound bed and discard dressing.

Also available for toes



WHEN TO USE

ADAPTIC™ DIGIT Dressing is indicated to manage a range of mild to severe digit injuries.

The dressing can be used to manage:

- Lacerations and abrasions
- Traumatic injuries
- Nail extractions
- Suture lines

ADAPTIC™ DIGIT Dressing can be used both as a primary and secondary dressing².

Good to know...

In a clinical evaluation study

96%

of patients considered ADAPTIC™ DIGIT easy or very easy to remove¹

ADAPTIC™ DIGIT		
Size	Dressings per box	SKU
Small 3/4 in (2cm)	10	MAD003U
Medium 1 in (2.4cm)	10	MAD013U
Large 1 1/8 in (2.8cm)	10	MAD023U
Extra large 1 1/4 in (3cm)	10	MAD042U

ADAPTIC™ DIGIT TOE		
Size	Dressings per box	SKU
Large 1 1/8 in (2.8cm)	10	MAD062U

To learn more about the benefits of ADAPTIC™ DIGIT Dressing contact your KCI representative or call **800.275.4524** or visit acelity.com.

References
1. Whittaker S. A dressing for the occasion? A comparative trial of two dressings for digit injuries. *Professional Nurse Magazine*, 1994 Aug. 729-732.
2. Hargreaves, J. The Use of a Non-Adhering Digit Dressing on wounds in a Primary Care Setting. Paper presented EWMA, Krakow, Poland, May 2018.

NOTE: Specific indications, contraindications, warnings, precautions and safety information may exist for Systagenix and KCI (Acelity companies) products. Please consult a healthcare provider and product instructions for use prior to application. Rx only.

Copyright 2018 KCI Licensing, Inc. All rights reserved. Unless otherwise designated, all trademarks are proprietary to KCI Licensing, Inc., its affiliates and/or licensors. PRA-PM-US-00794 • LIT 29-B-297 (12/18)

