

Biatain® Alginate

Application and removal

How to apply Biatain® Alginate Dressing

Abbreviated instructions for use. Please refer to product labeling for complete product instructions for use.

Preparation

1. Rinse the wound with the physiological saline or tap water. Gently dry the skin around the wound.
2. Biatain Alginate Dressing can be cut to fit the wound. If you need to cut the dressing you are advised to use a sterile pair of scissors.
3. Biatain Alginate Dressing should be used with a secondary cover dressing e.g. Comfeel® Plus Transparent Dressing or Biatain® Foam Dressing depending on the clinical condition of the wound.
4. Biatain Alginate Dressing should be changed when the secondary wound dressing is changed.



Biatain Alginate Dressing should not be left on for more than 7 days.

How to remove

When exudate levels have decreased and wound healing is progressing, Biatain Alginate Dressing may be discontinued and replaced by another suitable dressing (e.g. Biatain Foam Dressing or Comfeel Plus Ulcer Dressing) to ensure continuation of wound healing.

5. Biatain Alginate Dressing can easily be removed by rinsing with a saline solution. Remove with forceps without causing pain or trauma to the newly formed tissue.

How to apply Biatain® Alginate Rope

Preparation

1. Rinse the wound with the physiological saline or tap water. Gently dry the skin around the wound.
2. Biatain Alginate Rope can be cut to fit small or narrow wounds. If you need to cut the dressing you are advised to use a sterile pair of scissors.
3. Biatain Alginate Rope should be applied evenly/loosely into the wound without packing it. This will allow the alginate fibre to form a moist gel and conform to the size of cavity. Biatain Alginate Rope can be soaked in physiological saline prior to application. During absorption the rope dressing will not expand from its original size, but adjust to the cavity.
4. Biatain Alginate Rope should be used with a secondary cover dressing e.g. Comfeel Plus Transparent Dressing or Biatain Foam Dressing depending on the clinical condition of the wound. Biatain Alginate Rope should be changed when the secondary wound dressing is changed.



How to remove

When exudate levels have decreased and wound healing is progressing, Biatain Alginate Rope may be discontinued and replaced by another suitable dressing (e.g. Biatain Foam Dressing or Comfeel Plus Ulcer Dressing) to ensure continuation of wound healing.



5. Biatain Alginate Rope can easily be removed by rinsing with a saline solution. Remove with forceps without causing pain or trauma to the newly formed tissue.

Biatain® Alginate Dressings



Usage 🌊-🌊🌊 Exudate Level

Biatain Alginate is indicated for the management of moderately to heavily exuding wounds. Biatain Alginate can be used to facilitate exudate management and local haemostasis during the wound healing process as an external wound dressing in wounds like pressure ulcers, venous leg ulcers, arterial ulcers, diabetic foot ulcers, donor sites or traumatic wounds. Biatain Alginate can be used for diabetic foot ulcers under the discretion of a physician. Biatain Alginate can be used for sloughy wounds and cavity filling.

Biatain® dressings are single-use only. See the device manual for detailed information regarding complete instructions, contraindications, warnings, precautions and potential adverse events. For further information call Coloplast Corp. at 1-800-533-0464 and/or consult the company website at www.coloplast.us.