

Endoform® Natural Dermal Template and Endoform® Antimicrobial Dermal Template Application Instructions

Endoform® wound dressings are an extracellular matrix which supports all phases of healing.

Endoform® dressings contain:

- Natural intact collagen scaffold
- Secondary molecules that are important during healing
- Only natural tissue components

Endoform® has significant advantages over traditional collagen dressings and maintains all the benefits

- Supports healing in acute and chronic wounds
- Reimbursed as a collagen dressing (A code)
- Widely accessible to wound care clinicians and patients needing wound care from day one

Indications for use:

Endoform® Natural Dermal Template and Endoform® Antimicrobial Dermal Template are indicated for the management of wounds including, partial and full thickness wounds, pressure ulcers, venous ulcers, diabetic ulcers, chronic vascular ulcers, tunneled/undermined wounds, surgical wounds (donor sites, grafts, post Moh's surgery, post laser surgery, podiatric, and wound dehiscence), traumatic wounds (abrasions, lacerations, first and second degree burns, and skin tears), and draining wounds.



1. Prepare:

Cleanse the wound bed.



2. Apply:

Trim dressing to fit the wound. If needed, multiple sheets can be used to cover the surface of the wound. Place the dry dressing in the wound and hydrate with exudate or saline solution until moistened.



3. Cover:

Select appropriate secondary dressing based on type of wound and amount of exudate. If little to no exudate, apply a hydrogel to maintain appropriate moisture balance. Example: Hydrofera Blue dressing (shown).



4. Secure:

If needed, secure cover dressing with rolled gauze. The **Endoform®** may be used in conjunction with compression therapy or negative pressure wound therapy.

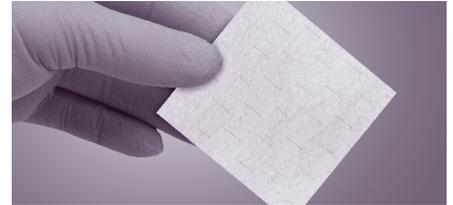


5. Reapply:

Duration of treatment and reapplication is determined by the physician and depends upon the wound type and conditions. Reapply when the dermal template has been incorporated into the wound. It is recommended to assess the wound within 72 hours. **Endoform®** Dermal Template may last up to 7 days in the wound, depending on the wound environment. It is not necessary to remove any residual **Endoform®** during dressing changes or reapplication of product.

Endoform® Natural Dermal Template - Fenestrated

Stock no.	Product Size	Quantity/Box	HCPCS
529312	2x2" (5cm x 5cm) fenestrated	10	A6021
529314	4x5" (10cm x 12.7cm) fenestrated	10	A6022


Endoform® Natural Dermal Template - Non Fenestrated

Stock no.	Product Size	Quantity/Box	HCPCS
529311	2x2" (5cm x 5cm) non-fenestrated	10	A6021
529313	4x5" (10cm x 12.7cm) non-fenestrated	10	A6022

Endoform® Antimicrobial Dermal Template - Fenestrated

Stock no.	Product Size	Quantity/Box	HCPCS
629312	2x2" (5cm x 5cm) fenestrated	10	A6021
629314	4x5" (10cm x 12.7cm) fenestrated	10	A6022


Endoform® Antimicrobial Dermal Template - Non Fenestrated

Stock no.	Product Size	Quantity/Box	HCPCS
629311	2x2" (5cm x 5cm) non-fenestrated	10	A6021
629313	4x5" (10cm x 12.7cm) non-fenestrated	10	A6022

For more information, see product instructions included with the product.

Additional Instructions

Rx Only. Prior to use, be sure to read the entire Instructions for Use package insert supplied with the product.

For product questions, sampling needs, or detailed clinical questions concerning our products in the US, please call 1-860-337-7730

HCPCS are for reference only and subject to change.

Endoform® is a registered trademark of Aroa Biosurgery Limited.



Endoform® Dermal Template is marketed in the USA by Appulse. www.appulsem.com



Manufactured for:
AROABIOSURGERY INC

www.aroabio.com

Hydrofera Blue is a trademark of Hydrofera LLC.
Endoform® is a trademark of Aroa Biosurgery Limited.
©2018 Aroa Biosurgery Limited

MKT 1445.01