



**Gentell**  
2701 Bartram Road  
Bristol, PA 19007  
215-788-2700  
www.gentell.com

## Instructions for Use

**Product:** GENTELL Dermatell Hydrocolloid 4" x 4" Dressing (2.5" pad)  
GENTELL Dermatell Hydrocolloid 6" x 6" Dressing (4.5" pad)

**Product Code:** GEN-10200 4" x 4"  
GEN-10400 6" x 6"

**Date Prepared:** March 2018

**Version:** 1

### Indications:

Gentell Dermatell Hydrocolloid Dressing consists of a hydrocolloid wafer attached to a water resistant adhesive border that is flexible and conforms easily to the body. Soft, pliable Dermatell dressings enhance patient comfort and protection while promoting moist wound healing by drawing out exudate and enabling natural debridement. Gentell Dermatell should remain on the patient for at least three days and applied on Stage 2 or Stage 3 wounds.

### Contraindications:

Dermatell is not indicated for use on heavily exuding wounds, Stage 4 pressure ulcers, infected wounds and third-degree burns.

### Precautions:

The dressing may adhere if used on very lightly exuding wounds. If the dressing is not easily removed, moisten with sterile water or saline prior to removal. The dressing's performance may be impaired by excess use of petrolatum-based ointments.

**Site Preparation:**

Clean the wound with Gentell Wound Cleanser and gently dry the skin surrounding the wound site.

**Dressing Selection:**

Determine if the dressing is an appropriate size based on the dimensions of the wound.

**Dressing Application:**

As a primary dressing, apply directly to the wound surface. As a secondary dressing, apply directly over primary treatment.

**Dressing Change and Removal:**

Change the dressing every three days, or when there is leakage from the wound or when the dressing gets dirty. To remove the dressing, apply a saline solution to loosen the edges and gently lift off the wound site.

**Storage:**

Store in a dry place at ambient temperature.

**Do not re-use.**

**Do not use if individual package is damaged / opened.**

**The expiration date of this product is printed on the packaging.**