


Brava® Ostomy Support Belt

Step-by-step sizing guide featuring the Brava Ostomy Support Belt Measuring Tool

The **Brava® Ostomy Support Belt** offers comfortable support for those with an outward area and is designed to prevent hernias post-operatively.

Ensuring proper sizing and fit is imperative for the product user to receive the appropriate support.

The **Brava Ostomy Support Belt Measuring Tool** features both metric and imperial measurements as well as color-coded size ranges, allowing for easy sizing.

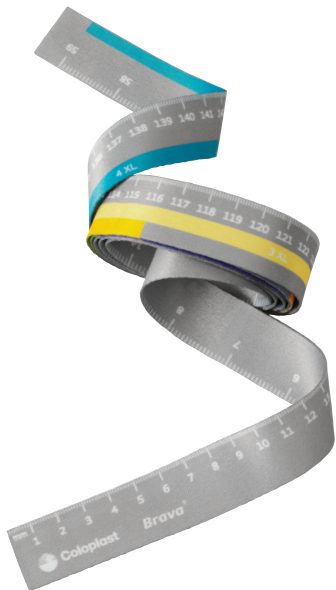

Step 1: Product user should lay down for 5-15* minutes prior to measuring to allow the hernia to reduce in size. This will help ensure the belt is sized to provide appropriate support *prior* to the hernia.

Step 2: While laying down, using the Brava Ostomy Support Belt Measuring Tool, or a measuring tape, **measure around the widest part of the abdomen.** The tape should be taught, but not squeezing the abdomen.

Step 3: Using the color-coded size ranges on the Brava Ostomy Support Belt Measuring Tool, determine the appropriate size. (If using a measuring tape, identify the proper belt size using the table to the right).

Note: If the product user falls between sizes (in an overlapping region) the product user can size down for stronger support and size up for increased comfort.

Learn more at www.coloplast.us/BravaOstomySupportBelt

*Time will depend on the size of the hernia

Please refer to product labeling for complete product instructions for use, contraindications, warnings, precautions and adverse events.



Item number	Size	Sizing	Unit in package
12003	S	29-32"	1
12004	M	31-35"	1
12005	L	34-39"	1
12006	XL	37-42"	1
12007	2XL	40-46"	1
12008	3XL	44-51"	1
12009	4XL	50-58"	1