

# ESENTA™

## Frequently Asked Questions



### What is the meaning behind the Esenta™ brand?

Esenta™ comes from the concept that accessories are essential to everyday life for people living with stoma and hard-to-heal wounds.

### How does the Esenta™ Skin Barrier work?

The protective silicone barrier bonds gently to the skin surface to create a fine, durable, protective silicone film. This breathable barrier protects the skin for up to 72 hours<sup>1</sup> from damage caused by adhesives, body waste, and enzymes.

### Can the Esenta™ Skin Barrier be used together with the Esenta™ Adhesive Remover?

Yes, the combination of both of these products is very common.

### Are the Esenta™ Skin Barrier and Adhesive Remover products flammable?

Yes, all liquid silicones used by manufacturers with products in the No Sting category are flammable. The Skin Barrier has a long history of safe use and will only combust if safety instructions are ignored or the product is misused. The accompanying labelling and Instructions For Use provide warnings and cautions.

### Does the Esenta™ Skin Barrier and Adhesive Remover contain alcohol?

No, they do not contain alcohol, the formulation is a silicone blend. Silicone blends do not sting on application.

### How do Esenta™ Adhesive Remover products work?

**Esenta™ Adhesive Remover Spray** - The fluid in the spray is designed to 'track' between the adhesive and the skin, creating a layer of silicone. This silicone layer then acts as a release agent to release the bond that is currently there, so the adhesive falls away from the skin without any trauma. The silicone then evaporates in seconds, without leaving any film or coating, and a new appliance can be applied (if required) without first needing to clean the skin.

**Esenta™ Adhesive Remover Wipes** - The wipe is designed to remove any adhesive residue left on the skin. The silicone from the wipe then evaporates in seconds, without leaving any film or coating, and a new appliance can be applied (if required) without first needing to clean the skin.

### How frequently can Esenta™ Skin Barrier be used?

Reapplication is recommended every 12-72 hours, dependent upon skin surface conditions. Safe and effective even with frequent use. Should be replaced at every dressing or appliance change to maintain complete protection.



Ostomy Care