


I CAN *live free*

ESTEEM™ Accuseal™ Tap

YOU ASKED. WE DELIVERED.
CLEAR BENEFITS.


New Esteem™+ One-piece Cut-To-Fit Urostomy Pouch with Durahesive™ Skin Barrier and Accuseal™ Tap

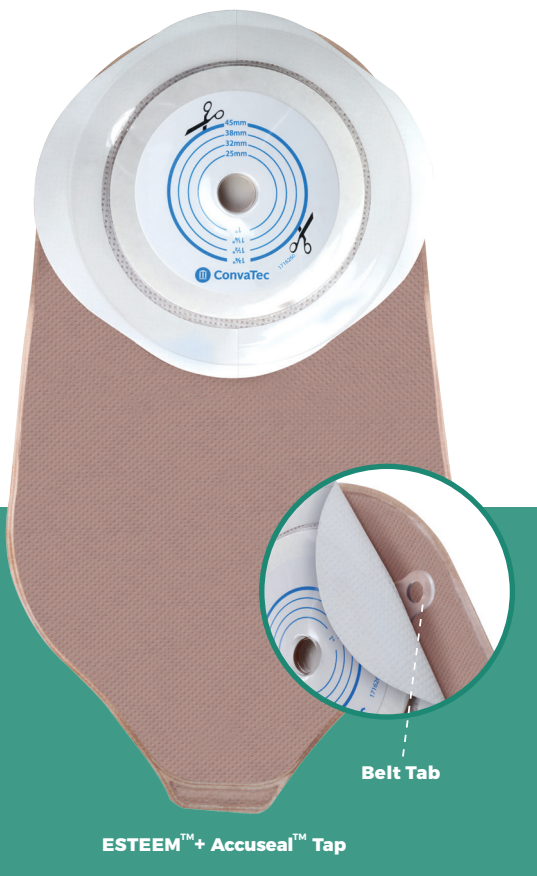


ESTEEM™+ Accuseal™ Tap



Ease-of-use: Red Marking Indicates open (drain) position

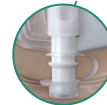

Outlet with tap and cap & anti-reflux valve

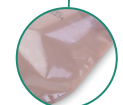

Transparent clothing-side film



Cutting guide sticker

Key Benefits Include:

- **Durahesive™ Skin Barrier**
 - Designed for skin health and security to help minimise leakage
- **Cut-To-Fit**
 - Tailored to individual stoma size and shape
- **Belt Tabs**
 - Soft flexible belt tabs for secure attachment of pouch to ostomy belt
- **Body-Side Flocking**
 - For comfort and discretion
- **Transparent Clothing-Side Film**
 - For accurate positioning of pouch and observation of stoma and output
- **Accuseal™ Tap**
 - Quickly and easily drains liquid output
 - Swivel mechanism – open and closed positions
 - Red marking indicates open (drain) position
 - Designed for connection to night drainage adaptor
- **Anti-reflux Valve**
 - Helps to prevent backflow of output


ESTEEM™ Accuseal™ Tap

Cut-To-Fit Skin Barrier


Skin Preparation

Your hands and the skin surrounding the stoma must be clean, dry and free from oily substances before applying.

Skin Barrier Preparation

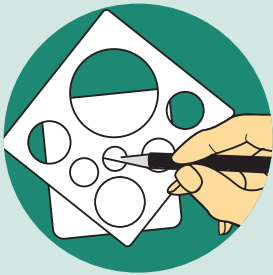


Trace the stoma outline onto the release liner. The starter hole is ½ inch (13mm) in diameter and can be cut to the same size and shape as the stoma, up to a maximum diameter of 1¾ inch (45mm).

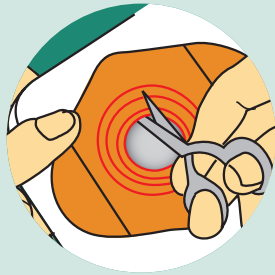


Cut the skin barrier just outside the tracing. Do not cut beyond the outermost circle. Run your finger along the inside of the hole along the cut edge to smooth any sharp edges.

Applying the Pouch

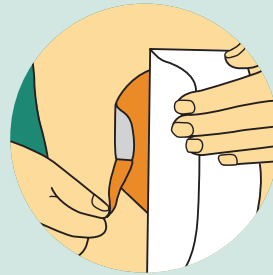


Remove the release liner from the barrier. Center the opening of the skin barrier over the stoma. Apply the skin barrier against the skin around the stoma for 30 seconds. Remove the release liner from the fabric border.

Attachment of Ostomy Belt

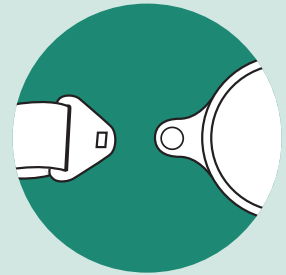


Smooth in place and remove wrinkles. If a belt is required for additional support or security, attach the belt to the Lugs provided and adjust to a comfortable fit. Wear as long as comfortable and secure. Wear time may vary.

Closing the Pouch

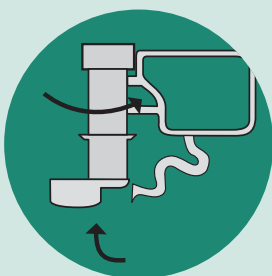


Make sure that the Accuseal™ Tap is rotated into the closed position and the plug is in the tap stem.

Emptying the Pouch and Connection to Night Drainage System

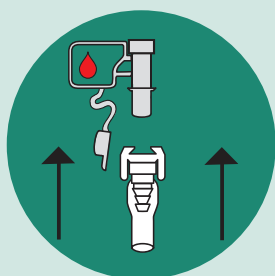

To drain the pouch make sure that the Accuseal™ Tap is rotated into the open position. To attach to a night drainage connector, use the adapter provided, and ensure that the Accuseal™ Tap is in the open position.

Removing the Pouch and Disposal

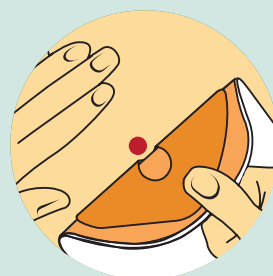

A water-soluble adhesive remover is recommended if the pouch is difficult to remove. Please follow all local disposal regulations.

Esteem™+ One-piece Cut-to-Fit Urostomy

Product Code	Cutting Range	Pack Size
421631	½ in. - 1 ¾ in. / 13-45mm	10

For more information, call us on -

XXXX XXX XXX

Please refer to the package insert for complete instructions for use.